

Second-generation fully automated Elecsys cerebrospinal fluid immunoassays demonstrate high precision, reproducibility, and sample stability suitable for clinical use to aid Alzheimer's disease diagnosis

P-69299

Sandra Rutz,¹ Ekaterina Manuilova,¹ Laura Mueller-Huebner,¹ Timo Grimmer,² Jennifer Powers Carson,³ Robert H. Christenson,⁴ Maria Karpova,¹ Carla Winter,¹ Charlotte E. Teunissen⁵

¹Personalized HealthCare Solutions, Roche Diagnostics GmbH, Penzberg, Germany; ²Department of Psychiatry and Psychotherapy, Klinikum rechts der Isar, Technical University of Munich, School of Medicine, Munich, Germany; ³Core Lab for Clinical Studies, Washington University School of Medicine, St Louis, MO, USA; ⁴Department of Pathology, Department of Medical and Research Technology, University of Maryland School of Medicine, Baltimore, MD, USA; ⁵Neurochemistry Laboratory, Department of Clinical Chemistry, Vrije Universiteit (VU) University Medical Center, Amsterdam, Netherlands

Introduction

- Cerebrospinal fluid (CSF) amyloid-beta (1-42) (Aβ42) and phosphorylated Tau 181 (pTau181) biomarker analyses are recommended alongside cognitive evaluation to aid diagnosis of Alzheimer's disease (AD).¹
 - Aβ42 peptide measured in the CSF is inversely correlated with amyloid positivity detected using positron emission tomography (PET).²
 - CSF Aβ42 and pTau181 can discriminate between AD and age-matched controls and predict progression to AD dementia in patients with mild cognitive impairment (MCI).³
 - Amyloid positivity as determined by the CSF pTau/Aβ42 ratio strongly correlates with amyloid PET positivity.⁴
- For clinical use, CSF biomarker assays should demonstrate high precision, reproducibility, and sample stability.
- Elecsys® β-Amyloid (1-42) CSF II and Elecsys Phospho-Tau (181P) CSF (Roche Diagnostics International Ltd, Rotkreuz, Switzerland) are electrochemiluminescence immunoassays for the *in vitro* quantitative determination of CSF levels of Aβ42 and pTau181, respectively.

Results

Precision

- Repeatability CVs were <2.4% for Aβ42, <2.0% for pTau181, and <2.4% for pTau181/Aβ42 ratio (Table 1).
- Intermediate precision CVs were <3.4% for Aβ42, <3.8% for pTau181, and <3.9% for pTau181/Aβ42 ratio (Table 1).

Reproducibility

- Site-to-site reproducibility CVs were <5.1% for Aβ42, <3.6% for pTau181 and <6.5% for pTau181/Aβ42 ratio (Table 2).
- For Aβ42, lot-to-lot reproducibility CVs were <5.5%, except for one sample pool (mean concentration: 322 pg/mL; SD: 34.1 pg/mL [acceptance criterion: ≤48.0 pg/mL]; Table 3). Lot-to-lot reproducibility CVs were <3.7% for pTau181 and <6.0% for pTau181/Aβ42 ratio (Table 3).

Sample stability

- At maximum sample storage duration of 8 days at room temperature (25°C) (Figure 1) and 15 days at 2–8°C (Figure 2), mean percent recoveries of 95% and 98%, respectively, were observed for Aβ42, and 97% and 99%, respectively, were observed for pTau181.
- Sample stability assessment at -20°C is ongoing.

Table 1. Repeatability and intermediate precision of the Elecsys β-Amyloid (1-42) CSF II and Elecsys Phospho-Tau (181P) CSF immunoassays.

Biomarker	Sample	Repeatability			Intermediate precision	
		Mean (pg/mL)	SD (pg/mL)	CV (%)	SD (pg/mL)	CV (%)
Aβ42	PreciControl 1	515	3.88	0.8	12.2	2.4
	PreciControl 2	1767	29.9	1.7	58.3	3.3
	CSF 1	302	3.28	1.1	5.67	1.9
	CSF 2	793	7.88	1.0	17.9	2.3
	CSF 3	1027	11.5	1.1	23.1	2.2
	CSF 4	1305	14.7	1.1	35.7	2.7
	CSF 5	1243	15.3	1.2	28.5	2.3
pTau181	CSF 6	2374	40.3	1.7	70.7	3.0
	CSF 7	2317	52.4	2.3	69.0	3.0
	PreciControl 1	14.2	0.275	1.9	0.526	3.7
	PreciControl 2	47.7	0.788	1.7	0.974	2.0
	CSF 1	15.5	0.263	1.7	0.389	2.5
	CSF 2	22.3	0.268	1.2	0.434	2.0
	CSF 3	26.3	0.398	1.5	0.530	2.0
pTau181/Aβ42 ratio	CSF 4	33.3	0.579	1.7	0.616	1.9
	CSF 5	58.4	0.764	1.3	0.979	1.7
	CSF 6	108	1.54	1.4	1.79	1.7
	CSF 7	115	1.53	1.3	1.94	1.7
	Ratio sample 1	0.021	0.0005	2.3	0.0008	3.8
	Ratio sample 2	0.028	0.0006	2.0	0.0009	3.1
	Ratio sample 3	0.038	0.0006	1.6	0.001	2.8
pTau181/Aβ42 ratio	Ratio sample 4	0.041	0.0008	2.0	0.001	2.8
	Ratio sample 5	0.054	0.0009	1.6	0.001	2.5

Table 2. Site-to-site reproducibility of the Elecsys β-Amyloid (1-42) CSF II and Elecsys Phospho-Tau (181P) CSF immunoassays.


Biomarker	Sample	Mean concentration (pg/mL)	SD (pg/mL)	CV (%)
Aβ42	PreciControl 1	590	18.6	3.2
	PreciControl 2	1751	60.2	3.4
	CSF level 1	397	8.7	2.2
	CSF level 2	856	19.2	2.2
	CSF level 3	1120	21.2	1.9
	CSF level 4	1291	31.4	2.4
	CSF level 5	1358	28.4	2.1
pTau181	CSF level 6	2231	111	5.0
	CSF level 7	2448	61.4	2.5
	PreciControl 1	15.4	0.540	3.5
	PreciControl 2	54.2	1.58	2.9
	CSF level 1	16.5	0.565	3.4
	CSF level 2	22.3	0.682	3.1
	CSF level 3	27.8	0.959	3.4
pTau181/Aβ42 ratio	CSF level 4	33.6	0.956	2.8
	CSF level 5	61.2	1.71	2.8
	CSF level 6	118	3.12	2.6
	Ratio sample 1	0.018	0.0008	4.2
	Ratio sample 2	0.031	0.0019	6.1
	Ratio sample 3	0.049	0.0018	3.6
	Ratio sample 4	0.047	0.0014	3.0
pTau181/Aβ42 ratio	Ratio sample 5	0.047	0.0014	3.1
	Ratio sample 6	0.020	0.0013	6.4
	Ratio sample 7	0.024	0.0014	5.6

Table 3. Lot-to-lot reproducibility of the Elecsys β-Amyloid (1-42) CSF II and Elecsys Phospho-Tau (181P) CSF immunoassays.

Biomarker	Sample	Mean concentration (pg/mL)	SD (pg/mL)	CV (%)
Aβ42	PreciControl 1	546	29.3	5.4
	PreciControl 2	1892	66.5	3.5
	HSP 1	322	34.1	10.6
	HSP 2	886	46.0	5.2
	HSP 3	1042	45.8	4.4
	HSP 4	1235	55.4	4.5
	HSP 5	1447	57.2	4.0
	HSP 6	2391	108	4.5
pTau181	HSP 8	2694	126	4.7
	PreciControl 1	14.1	0.394	2.8
	PreciControl 2	48.1	1.42	3.0
	HSP 1	15.5	0.552	3.6
	HSP 2	20.9	0.608	2.9
	HSP 3	26.2	0.679	2.6
pTau181/Aβ42 ratio	HSP 4	30.2	0.736	2.4
	HSP 5	59.3	1.28	2.2
	HSP 6	115	2.22	1.9
	HSP 6	0.0205	0.00111	5.4
	HSP 7	0.0244	0.00121	5.0
	HSP 8	0.0361	0.00214	5.9
pTau181/Aβ42 ratio	HSP 9	0.0389	0.00182	4.7
	HSP 10	0.0373	0.00146	3.9


HSP, human sample pool.

Figure 1. Regression analysis of sample stability for 8 days at room temperature (25°C).

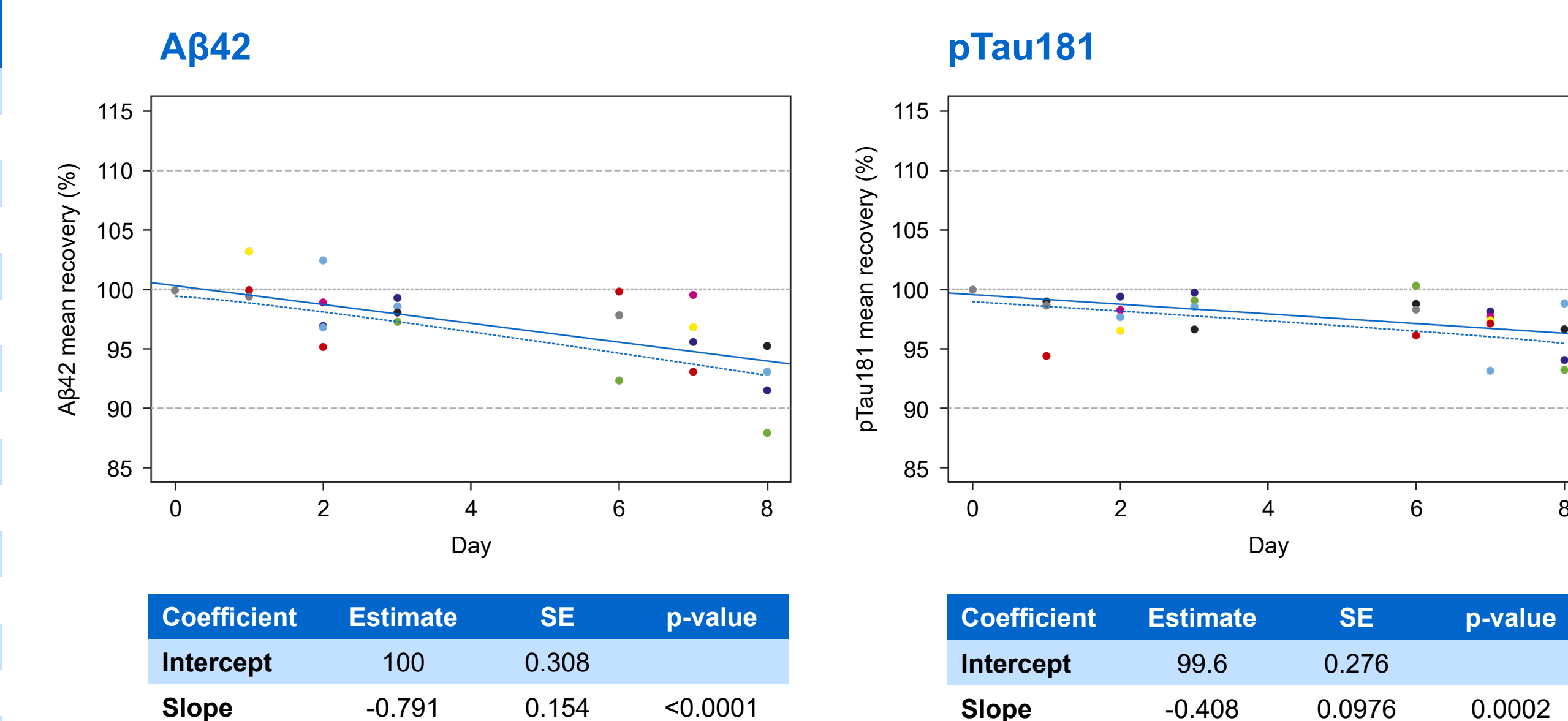

Identically colored points denote data from the same subject. Blue dotted line denotes the limit of the one-sided 95% confidence interval for the regression line, derived using Jackknife approach. SE, standard error.

Figure 2. Regression analysis of sample stability for 15 days at 2–8°C.

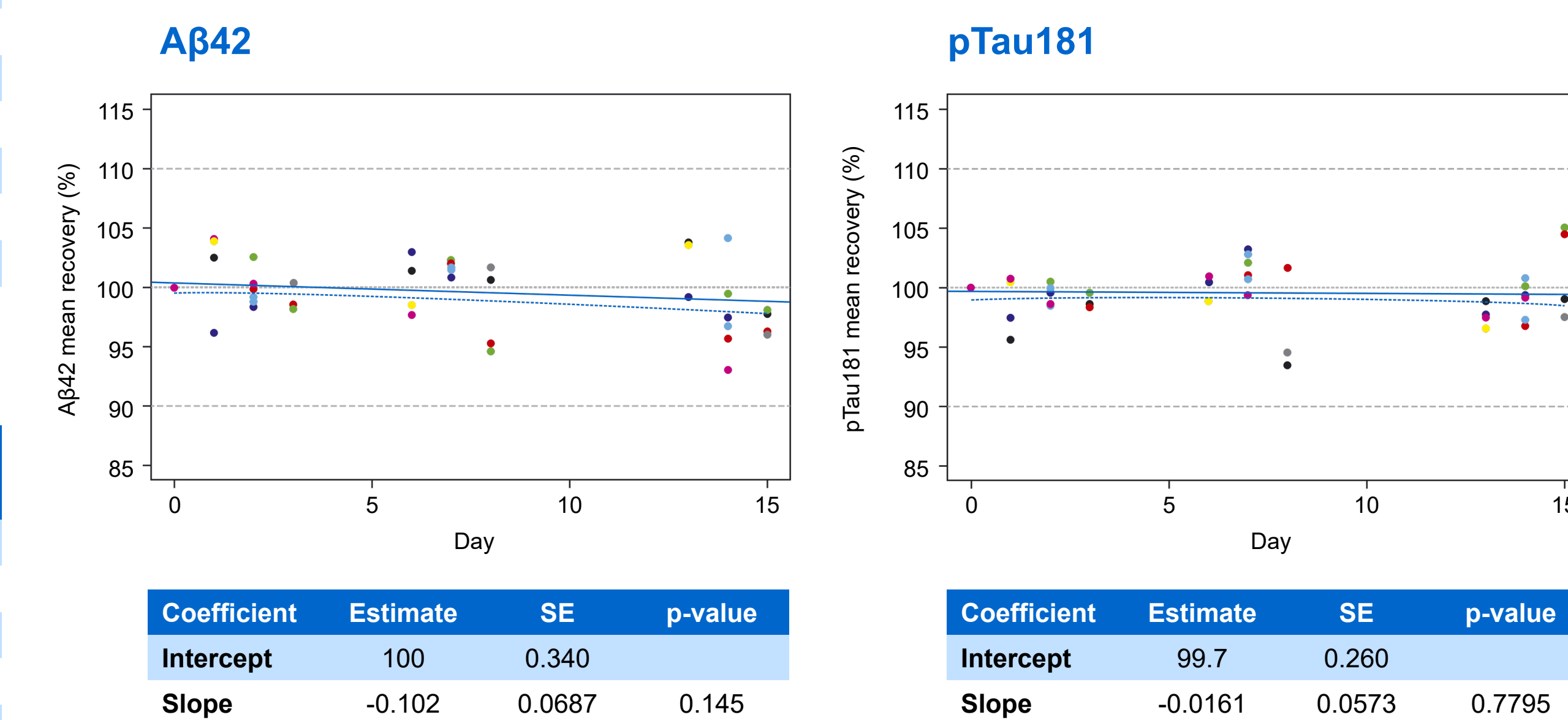

Identically colored points denote data from the same subject. Blue dotted line denotes the limit of the one-sided 95% confidence interval for the regression line, derived using Jackknife approach. SE, standard error.

Conclusions

- The Elecsys β-Amyloid (1-42) CSF II and Elecsys Phospho-Tau (181P) CSF immunoassays demonstrated high precision and reproducibility, supporting their clinical use in AD diagnosis.
- Conservative recommendations for CSF sample storage are 5 days at room temperature (25°C) or 14 days refrigerated at 2–8°C.

References

- Jack Jr. CR, et al. *Alzheimers Dement* 2018;14:535–62.
- Lewczuk P, et al. *Adv Med Sci* 2015;60:76–82.
- Mattsson N, et al. *JAMA* 2009;302:385–93.
- Campbell MR, et al. *Alzheimers Dement (Amst)* 2021;13:e12190.

Disclosures

This study was sponsored by Roche Diagnostics International Ltd (Rotkreuz, Switzerland). Third-party medical writing assistance, under the direction of the authors, was provided by Keerthi Sivan, MSc, of Ashfield MedComms (Macclesfield, UK), an Inizio company, and was funded by Roche Diagnostics International Ltd (Rotkreuz, Switzerland). COBAS, COBAS E, and ELECSYS are trademarks of Roche. The Elecsys β-Amyloid (1-42) CSF II and Elecsys Phospho-Tau (181P) CSF immunoassays are not approved for clinical use in the US.



Objectives

- To evaluate the precision and reproducibility of the second-generation fully automated Elecsys CSF immunoassays on the cobas e 601 analyzer (Roche Diagnostics International Ltd).
- To assess the effect of storage conditions on CSF sample stability.

Methods

- This multicenter study was conducted at four external sites (Amsterdam, Netherlands; Baltimore [MD] and St Louis [MO], USA; and Munich, Germany) and one internal site (Roche Diagnostics GmbH, Penzberg, Germany) between February–December 2021.
- Pooled/frozen spiked/native samples generated from uncharacterized CSF, sourced from third-party vendors and from residual routine clinical samples, and two PreciControl samples were analyzed using the Elecsys β-Amyloid (1-42) CSF II and Elecsys Phospho-Tau (181P) CSF immunoassays on the cobas e 601 analyzer, and pTau181/Aβ42 ratios were calculated from individual Aβ42 and pTau181 measurements.
- Precision was evaluated at the internal site per Clinical and Laboratory Standards Institute (CLSI) EP02-A3: two runs/day in duplicate over 21 days (n=84).
- Reproducibility was determined per CLSI EP05-A3 over 5 days using a single lot across three external sites (site-to-site; Amsterdam, Netherlands; Baltimore and St Louis, USA) and three lots (lot-to-lot) at one internal site.
- Coefficients of variation (CVs) and standard deviations (SDs) for repeatability, intermediate precision, and reproducibility were calculated using variance component analysis.
- Sample stability was determined at one external site (Munich, Germany) from fresh CSF samples, collected per manufacturer's instructions.
- Baseline measurements were taken <6 hours after lumbar puncture and follow-up measurements (from stored aliquots; 1–5 per subject) were taken at defined intervals for ≤8 days at room temperature (25°C; n=13), ≤15 days at 2–8°C (n=14), and for ≤13 weeks at -20°C (n≥25).
- Storage stability was determined using ordinary linear regression; mean percentage recoveries per subject and storage time were used for the analysis.